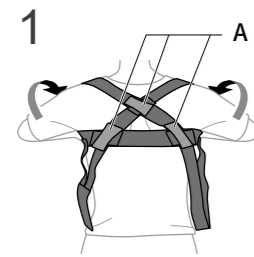
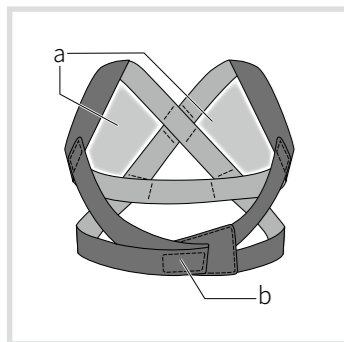
Style BX

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Purpose: This product is intended to help support your posture (it is not a medical device).

Safety Precautions

You should not use this product if any of the following apply.

- If you have a lumbar hernia or lumbar and back issues like sciatica
- If you are prone to allergies
- If you have been diagnosed with an aneurysm
- If you have suffered a circulatory disorder
- If you have injuries in the area the product will be used
- If you have sensory disorders in the area the product will be used
- If you are pregnant or may be pregnant
- If you are a child in your growth period

Even if none of the conditions above apply to you, please consult with your physician if you have any concerns about using the product.

- Ensure that you do not over-tighten the belt when using for the first time, and use for short periods of time (about 15 minutes) while seeing how it goes. In particular, those who are susceptible to rashes or itchiness should exercise caution.
- Do not use the product while sleeping.
- Do not augment, modify or attempt repairs on this product.
- Do not use this product if damaged.
- Please do not use this product on any body parts other than those specified.
- Ensure you use a product that is the proper size for you.

- Do not overtighten the product.
- Ensure that you put the product on over undergarments or clothing with sleeves.
- Ensure that the hook-and-loop fastener is not in direct contact with your skin.
- Stop use immediately if the product feels uncomfortable or if you sense something abnormal.
- Stop using the product if you feel pain due to extended periods of use.

Precautions for Use

- Ensure that the hook-and-loop fastener does not get affixed to clothing when putting the product on.

- Color transfer may occur if product gets wet, or if it rubs against other clothing or chairs.
- If the product gets dirty or warps, hand wash it (by pressing down) with a neutral detergent, remove excess water with a towel and dry it in the shade. Do not wring out the product.

- Do not wash the product with other items. Color loss or damage to other items due to the hook-and-loop fastener may occur.
- Avoid open flame, locations with high temperature or humidity, and direct sunlight when storing.
- Store out of reach of children and pets.
- Dispose of the product according to the regulations of your local government.

Product name	Style BX
Model Number	Small: (Black) BS-BX2234-S Medium: (Black) BS-BX2234-M Large: (Black) BS-BX2234-L
Size (cm, Chest measurement*)	Small: 62 - 79, Medium: 76 - 93, Large: 90 - 104 *Women should use their under bust measurement as a guide.
Materials	Main unit: Nylon 54% , Polyester 42%, Polyurethane 4% Parts (Back): Synthetic Leather
Items Included	Main unit (1), Accessories: Instruction manual (1)
Country of Origin	Japan

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